

Infusion Set


(NON VENTED)

DEVICE INTENDED USE:

Intravenous (IV) sets are used to deliver fluids and medications directly into a patient's bloodstream through a vein.

INSTRUCTIONS FOR USE:

- Close the Roller Clamp
- Remove Protector from Plastic Spike & Insert the Spike at the Top of the Bottle to its Full Length
- Squeeze the Drip Chamber Until it is Fluid Half
- Remove the Needle Protector Holding the Needle Upright
- Open the Roller Clamp & Allow the Solution to Pass until all Air
- Bubbles in the Tube are Removed Close the Clamp.
- Perform Vein Puncture and Regulate Flow by Opening the Flow Regulator.
- "Gravity Feed Only"

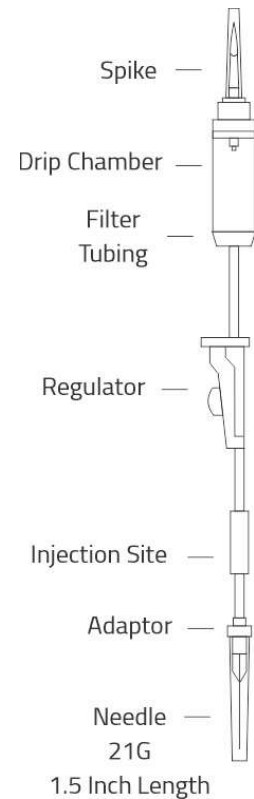



MRK HEALTHCARE PVT. LTD.

IFU

Infusion Set

(NON VENTED)

CAUTION:

- Do not use if the package is opened or Damaged.
- Do not use protective caps if are detached.
- Discard after single use.
- Not to be used for blood or blood components.
- Do not store at extreme temperatures & humidity.
- This device is used only for intravenous solutions.