

Nulife

Nulife is a global medical disposable solutions company with its footprint in 80+ countries and its headquarters in Mumbai, India.

With a workforce of over 2000 people worldwide, Nulife annually sells more than 300 million medical devices.

Triple Safety

Nulife Triple Safety is a brand that is committed to offering affordable healthcare solutions.

Powered by cutting-edge innovation, Nulife Triple Safety has disrupted the status quo in its sector through cost-effective healthcare products over the years.



Nulife Triple Safety Blood Transfusion Set

Product Information

The Nulife Triple Safety Blood Transfusion Set is used for intravascular, gravity-controlled transfusion of blood or blood elements.

Our Blood Transfusion Sets are designed for optimal procedural ease.



They use a smooth roller clamp to facilitate easy and controlled adjustment of fluids.

This ensures that the patients are comfortable during the transfusion.

Special Features

- Soft, kink-resistant, translucent tubing
- Clear chamber for visibility and access
- Specially designed roller (thumb) controller



TECHNICAL SPECIFICATION SHEET

Product Information

Product Name	Nulife Triple Safety Blood Transfusion Set
Material	PVC, ABC, PP, LDPE, SS Needle, DEHP-free PVC material
Sterilisation	Ethylene Oxide/Gamma as per customer requirement
Labelling	Shall comply with the corresponding labelling specification and customer requirement
Shelf Life	5 years from the date of manufacture
Storage Condition	Shall be stored in a cool, dry place and away from moisture, direct light & heat sources
Certifications	ISO 13485: 2016

Directions For Use:

1.1 How to use

- Close the control clamp.
- Remove the protector from the plastic spike and insert it into the outer opening of the Blood container.
- Hang the container upside down.
- Half-fill the drip chamber by gently squeezing and releasing the drip chamber.
- Pierce the air vent needle straight into the gum cap.
- Take off the vein-needle protector and open the clamp-roll slightly. Take out the Blood through the vein-needle to remove the air inside the tube completely.
- Shut the clamp-roll and perform a vein puncture. Fix the

vein-needle properly with tape.

- Regulate the flow by loosening the clamp-roll.
- Adjust to deliver approximately 20 drops/ml.

1.2 Cautions for Use

- The product is for single use only. Do not reuse.
- Sterility guaranteed till the pack is opened or damaged.

1.3 Storage Condition

- Do not store at extreme temperatures or expose to humidity and direct sunlight.

1.4 Expiration Date

- The shelf life of the Blood transfusion set is 5 years from the date of manufacture.

Disclaimer:

1. The description and specifications on the website are correct at the time of publishing.
2. In pursuit of continual improvement, we reserve the right to change the design and specifications of any of our products without prior notice.
3. All photographs, product designs and trademarks represented in this website are the sole intellectual property of MRK HEALTHCARE PVT. LTD.