

Nulife

Nulife is a global medical disposable solutions company with its footprint in 80+ countries and its headquarters in Mumbai, India.

With a workforce of over 2000 people worldwide, Nulife annually sells more than 300 million medical devices.

Triple Safety

Nulife Triple Safety is a brand that is committed to offering affordable healthcare solutions.

Powered by cutting-edge innovation, Nulife Triple Safety has disrupted the status quo in its sector through cost-effective healthcare products over the years.




Nulife Triple Safety Infusion Set

Product Information

Nulife Triple Safety Infusion Set uses superior quality tubing which is exceptionally smooth and minimises kinking. This ensures a consistent flow rate.

This set is employed for the controlled infusion of medications over extended periods.

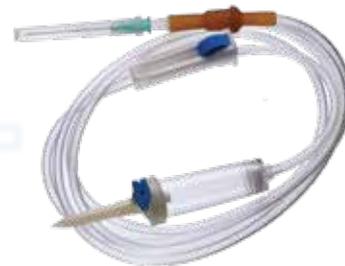

Spike	: Made up of ABS Material
Drip Chamber	: DEHP Free PVC material
Connector	: Luer Slip/Luer Lock/Y-Type
Injection Port	: Latex Injection Port
Needle	: With or Without Hypodermic Needle

Special Features

- Soft, kink-resistant, translucent tubing
- Clear chamber for visibility and access
- Specially designed roller (thumb) controller



TECHNICAL SPECIFICATION SHEET

Product Information

Product Name	Nulife Triple Safety Infusion Set
Sterilisation	Ethylene Oxide/Gamma as per customer requirement
Labelling	Shall comply with the corresponding labelling specification and customer requirement
Shelf Life	5 year Shelf Life
Storage Condition	Shall be stored in a cool, dry place and away from moisture, direct light & heat sources
Certifications	ISO 13485: 2016
Quality Standard	IS 12655, ISO 8536-4

Product Details

Model	Tube Length	Injection Site	Chamber	Needle	Fluid Filter	Drop/ml	Regulator	Pouch Packing
Eco	150 cm	Latex Tube	Without Built-in Airvent	21GX1.25"	Yes	20/ml	40mm	Polythene Poly Pouch
Deluxe	150 cm	Latex Bulb	Without Built-in Airvent	21GX1.25"	Yes	20/ml	40mm	Polythene Poly Pouch
NV-Deluxe	150 cm	Latex Bulb	Without Built-in Airvent	21GX1.25"	Yes	20/ml	40mm	Polypropylene Ribbon Pouch
Prime	150 cm	Latex Bulb	Without Built-in Airvent	21GX1.50"	Yes	20/ml	45mm	Polypropylene Ribbon Pouch
Classic	150 cm	Y-Site	Without Built-in Airvent	21GX1.50"	Yes	20/ml	45mm	Polypropylene Ribbon Pouch

Product Varieties

Eco	I.V. Set with latex tube
Deluxe	I.V. Set with latex bulb
NV-Deluxe	I.V. Set with latex bulb, available in ribbon pack
Prime	Vented with latex bulb, available in ribbon pack
Classic	Vented luer lock with 'Y' connector, available in ribbon pack

Packaging Details

Packing	Eco	Deluxe & NV-Deluxe	Prime & Classic
Pouch	1 pc	1 pc	1 pc
Inner Pack	25 pcs in Poly Bag	25 pcs in Poly Bag	25 pcs per Box
Master Carton	500 pcs (20 Poly Bags)	400 pcs (16 Poly Bags)	300 pcs (12 Inner Boxes)
20' Container	2,09,000 pcs (418 Cartons)	1,67,200 pcs (418 Cartons)	1,25,400 pcs (418 Cartons)
40' HQ Container	5,07,500 pcs (1015 Cartons)	4,06,000 pcs (1015 Cartons)	3,04,500 pcs (1015 Cartons)

Directions For Use:

1.1 How to use

- Close the control clamp.
- Remove the protector from the plastic spike and insert it into the outer opening of the solution container.
- Hang the container upside down.
- Half-fill the drip chamber with solution by gently squeezing and releasing the drip chamber.
- Pierce the air vent needle straight into the gum cap.
- Take off the vein-needle protector and open the clamp-roll slightly. Take out the solution through the vein-needle to remove the air inside the tube completely.
- Shut the clamp-roll and perform a vein puncture. Fix the

vein-needle properly with tape.

- Regulate the flow by loosening the clamp-roll.
- Adjust to deliver approximately 20 drops/ml.

1.2 Cautions for Use

- The product is for single use only. Do not reuse.
- Do not use with blood or blood components.
- Sterility guaranteed till the pack is opened or damaged.

1.3 Storage Condition

- Do not store at extreme temperatures or expose to humidity and direct sunlight.

1.4 Expiration Date

- The shelf life of the Infusion Set is 5 years from the date of manufacture.

Disclaimer:

- The description and specifications on the website are correct at the time of publishing.
- In pursuit of continual improvement, we reserve the right to change the design and specifications of any of our products without prior notice.
- All photographs, product designs and trademarks represented in this website are the sole intellectual property of MRK HEALTHCARE PVT. LTD.